BRODA®

Operating Manual

Sentinel Shower Commode Model

Distributor/Rep: This manual must be given to the caregiver responsible for this chair and its occupant.

Caregiver: Before using this chair, read this manual thoroughly and save for future reference.

Actual product may differ slightly from images in Operating Manual

Contents

FOREWORD.....	1
GENERAL INFORMATION.....	2
1 DEFINITIONS.....	3
2 SAFETY REQUIREMENTS.....	4
2.1 Training.....	4
2.2 Application.....	4
2.3 Shipping and Storage.....	4
2.4 Pre-Service Check.....	5
2.5 Hazards.....	5
2.5.1 Position of Chair - “Danger of Falling”.....	5
2.5.2 Location of Chair - “Danger of Tipping or Falling Objects”.....	5
2.5.3 Total Lock Wheel Brakes - “Danger of Falling”.....	5
2.5.4 Re-Positioning of Resident - “Danger of Clamping”.....	6
2.5.5 Unintended Movement - “Danger of Falling or Collision”.....	6
2.5.6 Improper Restraint Use - “Risk of Serious Injury”.....	6
2.6 Improper Use.....	6
2.7 Cleaning.....	7
2.8 Maintenance.....	8
2.9 Resident Specific Instructions.....	9
3 OPERATION AND MOVEMENT.....	10
3.1 Adjustable Seat Height.....	10
3.2 Back Height and Angle.....	11
3.3 Seat Depth.....	12
3.4 Caster Conversion.....	12
3.5 Mag Wheel Mounting Position.....	14
3.6 Anti-Tipper Height Adjustment.....	14
3.7 Commode Seat Mounting.....	15
3.8 Casters & Mag Wheels.....	16
3.9 Angle Adjustable Seat Dump.....	16
3.10 Removing and Adjusting Footrests.....	17
3.11 Removable Bed Pan & Basket (Option).....	18
3.12 Lateral Torso Support (Option).....	18
3.13 Accessories (Options).....	18
4 INSPECTION AND FUNCTIONAL TESTING.....	19
4.1 Inspection.....	19
4.2 Functional Testing.....	19
5 TECHNICAL INFORMATION.....	20
5.1 Specifications.....	20
5.2 Broda Sentinel Shower Commode Parts Schematic Legend- 2022.....	21
6 TROUBLESHOOTING AND REPAIR.....	21
7 MANUFACTURER WARRANTY.....	22

FOREWORD

Congratulations on your purchase of a Broda chair and thank you for your confidence in our company and products.

Broda assumes a leadership role in providing optimum re-positioning functions and mobility for residents of long-term care institutions. Our chairs do not look like traditional chairs and offer advantages unique to Broda. We are sure that after using your Broda chair, you will be convinced that your resident's quality of life will be greatly enhanced.

This manual will assist you in making the best use of the capabilities of your Broda chair and will ensure that you quickly become familiar with its operation.

After reading this manual, if you have any questions about the safe and effective operation of your Broda chair or accessories contact your local Broda representative or our Head Office Customer Service personnel for further assistance.

Head Office:
Broda
15455 Conway Road Suite 200
Chesterfield, Missouri 63017
Phone: (844) 552-7632
Fax: (844) 442-7632
E-mail: sales@brodaseating.com
URL: www.brodaseating.com

GENERAL INFORMATION

This document provides guidance on the safe and effective operation of the Broda chair.

Information in this manual must always be followed.

Anyone involved with the operation or maintenance of the Broda chair, including the resident's family members, must read this operating manual before using the chair.

The resident's primary caregiver is responsible for ensuring that anyone who is unfamiliar with, unwilling, or unable to adhere to the safety and operating instructions, is not permitted to operate or move the chair.

A copy of this instruction manual must always be available.

Broda accepts no liability for damages, injury or accidents caused by operating errors, improper maintenance, or disregard of the instructions in this manual, including any resident specific instructions.

Broda reserves the right to make changes to the specifications, dimensions, functions, or components of its products without notice. Product representations in this manual may vary from delivered products.

Each Broda chair has a unique identifying serial number that must be maintained on the chair as well as with any equipment records.

1 DEFINITIONS

“Broda” means Broda Enterprises Inc. doing business as Broda Seating.

“Broda chair” refers to model Sentinel Shower Commode.

“Long-Term Care Institution” refers to a nursing home, hospital, or other healthcare facility that provides health and personal care to its residents on a long-term basis.

“Resident” refers to an individual living in a long-term care institution under the care of professional caregivers.

“Professional Caregiver” refers to doctors, nurses, therapists, nursing aids, healthcare aids, and other specialists who work in a long-term care institution and provide health and personal care to its residents.

“Caregiver” refers to any person in a long-term care institution who is appropriately trained to provide care or services to the resident or the chair used by the resident and may include the resident’s family members or guardian.

“Transfer(s)” refers to the movement of a resident into or out of the chair with the assistance of their caregiver(s).

“Mechanical Transfer(s)” refers to the movement of a resident into or out of the chair with the assistance of their caregiver(s) using a patient lift or other assistive device that bares the weight of the resident.

2 SAFETY REQUIREMENTS

2.1 Training

Before the chair is put into service, this manual must be read thoroughly by the caregiver(s) directly responsible for the resident's care.

After the chair is put into service, this manual must be read thoroughly by any new caregivers prior to operating or moving the chair.

For the purpose of this manual, a resident's family member who shares responsibility for their care is considered a caregiver and is subject to the same competency before being permitted by the resident's primary caregiver to operate or move the chair.

Prior to first use, the customer must arrange for an in-service on the operation and safety requirements in this manual, must be given to the resident's caregivers by the local Broda representative who supplied the chair.

The primary caregiver must maintain a list of caregivers who have read this manual and who they have authorized to operate and move the chair.

The resident's primary caregiver is responsible for ensuring that anyone who is unfamiliar with, unwilling, or unable to adhere to the safety and operating instructions, is not permitted to operate or move the chair.

The operations of the chair must be performed by the resident's primary caregiver who is responsible for seating. All The operations and adjustments performed should be done in a manner to ensure the overall safety, comfort and well-being of the resident, caregiver and third party. All operations and adjustments required for the resident should be determined by the resident's primary caregiver who is responsible for seating.

2.2 Application

The suitability of a Broda chair must be determined by a qualified caregiver who is familiar with the seating needs of the intended resident. Any other use of the chair is excluded from possible liability claims.

The chairs are not explosion resistant and must not be used where there are flammable gases or liquids present (e.g., anesthetics, volatile solvents and cleaners, etc.)

Broda chairs are designed for use with specific Broda parts and accessories. The use of non-Broda parts or accessories with a Broda will void the warranty and is excluded from possible liability claims.

Broda chairs may only be used as described in this manual and with proper regard for recognized healthcare and workplace safety and accident prevention practices.

2.3 Shipping and Storage

Broda chairs should be shipped and stored in an upright position and not stacked higher than 3 boxes. No other materials should be shipped or stored on top of a Broda box. Broda boxes should not be placed on pallets.

Broda chairs should be shipped and stored at temperatures between -20°C and 40°C. Broda chairs should not be used until they are between 0°C and 30°C.

Broda Commode Shower Chairs should be wiped dry after use. Do not leave Broda chairs outdoors as it may cause the paint to peel.

Upon receipt, the shipping carton must be immediately examined for damage. Any damage should be noted on the delivery receipt and a request for inspection by the transportation company should be made. Next, the shipping carton should be opened, and the chair must be examined for concealed shipping damage. If the chair appears to be damaged, do not use the chair. File a concealed damage report with the transportation company.

2.4 Pre-Service Check

Broda chairs are delivered fully assembled. If the chair does not appear to be ready to use upon receipt, immediately contact your supplier and do not put the chair into service until any concerns have been resolved.

Visually inspect the chair for damage, missing parts, and loose fasteners prior to testing the chair's functions. Functional testing must be successfully completed after visual inspection and before use. These obligations apply to the chair's first use and to all subsequent uses (Section 4: Inspection and Functional Testing).

2.5 Hazards

2.5.1 Position of Chair - "Danger of Falling"

After a resident is transferred into a chair, assess the amount of tilt required. We recommend that the chair's seat be tilted sufficiently to prevent the resident from sliding or falling forward off the chair. The amount of seat tilt used should be determined by the resident's caregiver who is responsible for seating.

We recommend that the resident's feet be correctly positioned on the footrests and slightly to fully elevated to prevent the resident from sliding or falling forward off the chair. The amount of elevation used should be determined by the resident's caregiver who is responsible for seating.

2.5.2 Location of Chair - "Danger of Tipping or Falling Objects"

We recommend that when a resident has been moved to their destination, the chair is placed where the resident cannot reach handrails or other objects, fixed or movable. This is to prevent the resident from pulling the chair over or pulling themselves off the seating surface and to prevent the resident from pulling movable objects onto the chair and themselves.

We recommend that the chair be used in a supervised area to prevent untrained residents, caregivers, or third parties from unauthorized operation, movement, or unsafe actions such as sitting or leaning on the reclined back, elevated footrest, or the armrests. These actions, if not prevented, put the chair at risk of tipping or damage to the chair.

We recommend that a chair only be located on a level surface to minimize the risk of tipping over.

2.5.3 Total Lock Wheel Brakes - "Danger of Falling"

The special casters found on the Broda chair have total lock brakes which prevent the wheels from turning and swiveling. The brakes must always be applied when:

- 1) the chair is not in use
- 2) the chair is not being moved by the caregiver
- 3) a resident is being transferred (moved) into or out of the chair;

It is important to note that if the wheel locks are applied while the patient is in the chair, that the caregiver does not leave the patient unattended, especially those patients who have the capability or tendency to move the chair and/or those who may be agitated. This could cause harm to the patient if they attempt to move the chair while the wheel locks are applied.

Note that removing and attaching the footrest may be easier for the caregiver with the brakes applied

Failure to follow these instructions will unnecessarily increase the risk of serious falls by residents, caregivers, or third parties caused by the chair unintentionally moving.

2.5.4 Re-Positioning of Resident - “Danger of Clamping”

Broda chairs offer the benefits of seat tilt, back recline, leg rest/footrest elevation, and moveable arms. During the movement of any of these functions, the following safety measures must be observed:

- 1) The resident’s arms must be positioned on the armrests or inside the chair frame with their hands on their body or on.
- 2) The resident’s feet must be correctly positioned on the footrest.
- 3) All of the chair’s brakes have been applied.
- 4) Only one caregiver at a time attempts to operate the chair’s functions.
- 5) Only one chair function is operated at a time.
- 6) The rear wheels are in the trailing position, behind the chair frame.
- 7) The residents’ and caregivers’ body are clear of all pinch points before operating the chair’s functions.

Failure to follow these safety measures can put the residents’ or caregivers’ limbs at risk of injury. Residents who may be unaware of their body position or unable to maintain a safe body position are at the most risk of the danger from clamping and caregivers should be more cautious with these residents. A second caregiver may be required to ensure the safety of these residents during these operations.

2.5.5 Unintended Movement - “Danger of Falling or Collision”

We recommend Broda chairs for indoor use within a long-term care institution and where there is not enough slope to cause the chairs to move unaided. Chairs used where the surface is uneven or sloped are at risk of unintended movement and could become a serious danger to the resident, caregiver(s), or a third party. We recommend that Broda chairs are located away from stairwells, elevators, and exterior doorways within a long-term care institution.

2.5.6 Improper Restraint Use - “Risk of Serious Injury”

We recommend that alternatives to physical restraints be used with residents while seated in the chair except under the specific instructions of the resident’s primary caregiver and with permission of the resident’s family or guardian.

2.6 Improper Use

As outlined, the improper use of the chair is dangerous to the resident, caregivers, or third parties, and can consist of, but is not limited to the following:

- 1) Unauthorized operation of the chair’s functions.
- 2) Unauthorized movement of the chair.
- 3) Inappropriate use of the chair for a resident who has not been assessed by a qualified caregiver responsible for their seating.
- 4) Failure to frequently reposition the resident in the chair
- 5) Attempting to operate of multiple chair functions simultaneously by one or more caregivers.
- 6) Attempting to move the chair with the brake(s) applied.
- 7) Leaving the resident unattended in the chair near other objects.
- 8) Leaving an agitated resident in the chair in an unsupervised area.
- 9) Leaving a resident unattended.
- 10) Leaving a resident in a chair on a sloping surface.
- 11) Leaving a chair unattended on a sloping surface.
- 12) Using non-Broda accessories on the chair.
- 13) Using the chair at temperatures below 0°C.
- 14) Using the chair for any use other than its intended purpose.

2.7 Cleaning

Broda chairs should be wiped clean with soap and water. Diluted household strength ammonia or chlorine based cleaner, as well as a hospital grade cleaner may be used if necessary.

For best cleaning results, certain parts on the chair should be removed: Instructions for removal of the following parts are in this manual: Ensure that the entire chair including removable and non-removable parts is thoroughly cleaned.

Pay close attention to the frame, sockets, fasteners, parts and casters, for an even, thorough clean.

Remove parts for cleaning if desired:

- 1) Commode Seat
- 2) Vinyl Back support
- 3) Footrest
- 4) Commode Pan and Basket

Thoroughly clean all removable parts and padding, as well as the frame and components around the parts and padding that have been removed. To help maintain the function of the gas springs, we recommend keeping them lubricated after each use. A white food grade grease may be used. Solvents such as those found in spray lubricants should not be used.

Broda chairs should not be cleaned with petroleum-based cleaners. Any petroleum-based products that come in contact with any vinyl surface should be removed as quickly as possible.

Frequency and method of cleaning the chairs should be determined by facility infection control protocols. If visibly soiled, thoroughly clean the chair immediately as per the cleaning instructions in this manual.

The cleaning instructions in this manual are guidelines only. Results may vary under actual conditions. The information does not relieve the user of proper and safe use of the product and all cleaning agents and consideration for the overall cleaning maintenance of the chair. Cleaning and care instructions must be followed in conjunction with facility infection control protocols.

Broda will not accept warranty or liability claims on chairs that have not been cleaned according to the instructions or cared for in proper regard for patient, caregiver and third-party safety and hygiene. The use of certain agents may be harmful to the surface appearance and lifespan of the vinyl. Broda assumes no responsibility resulting from the use of such cleaning agents to the vinyl.

Touch up paint is available for the powder coated frame. Please call Broda's Customer Service Department if touch up paint is required. Do not leave Broda chairs outdoors as the frame is not UV protected. Leaving the chairs outdoors could cause the paint to peel.

2.8 Maintenance

The maintenance on a Broda chair will vary with the amount of use and the condition of the resident using the chair.

In regular use, after the initial inspection and functional testing, the chair should be inspected and tested bimonthly. We recommend visually inspecting for signs of wear, damage, loose or missing fasteners, and other safety concerns. Periodic testing of chair functions is also necessary. If a breakage, defect, or operational problem is detected, the chair must be successfully repaired, inspected and tested for function before it is returned to service.

The chair should be inspected and tested as often as each use if the chair is used by:

- 1) Aggressive or agitated residents.
- 2) Residents who have involuntary movements.
- 3) Residents weighing over 220 lbs.
- 4) A facility with irregular or sloped surfaces.
- 5) Any unauthorized person.

Do not use lubricants that contain solvents. Solvents may compromise any rubber components on the chair. If necessary, white food grade grease or lubricant can be used on any sliding components.

2.9 Resident Specific Instructions

The primary caregiver responsible for the specific resident’s seating shall add additional instructions necessary for the safe and effective use of the chair based on their professional experience and knowledge of the resident’s specific conditions and requirements.

Broda representatives are not professional caregivers and will not know the specific requirements of the individual using the chair. Broda relies on the knowledge, experience, and judgment of the resident’s professional caregiver to ensure the specific resident’s safety and comfort needs are satisfied while using the chair.

These instructions form an essential part of the Safety Requirements for using the chair and must be made available to all caregivers. Space is provided below to include these instructions.

Broda
15455 Conway Road Suite 200
Chesterfield, Missouri 63017
Phone: 1-844-552-7632
Fax: 1-844-442-7632
Email: sales@brodaseating.com
URL: www.brodaseating.com

3 OPERATION AND MOVEMENT

Safety measures as described in Section 1 must be observed when operating the chair's functions or moving the chair. Only authorized caregivers should attempt to operate or move a chair. The caregiver must use proper body mechanics when operating the chair and be prepared to support the weight of the resident while operating the chair.

Stainless steel fasteners are prone to 'cold welding' and will seize.


- When removing fasteners, check threads for galling and replace if necessary to minimize future risk of seizing.
- When installing fasteners, apply (reapply) an anti-seize compound (e.g., white food-grade grease) to threads.

3.1 Adjustable Seat Height

The Sentinel has two seat heights: 19.5" and 23.5". To adjust the seat height, follow these steps using 1/2" wrenches:

- 1) Remove one hex bolt from each side of the rear mag plate as shown in Fig 1.
- 2) Remove one hex bolt from each side of the front seat frame as shown in Fig 2.
- 3) Remove both wheel lock tubes by removing two hex bolts from each side as shown in Fig 3. This is to allow clearance to raise/lower the seat frame.
- 4) Raise/lower the seat frame to desired location and re-insert and tighten hex bolts using the holes circled in Figs 1, 2 and 4.
WARNING: It is highly recommended that two people complete this step.
- 5) Re-install both wheel lock tubes in same location

*Tip- temporarily removing the mag wheels first will make adjusting the seat height easier, but not required. To remove the mag wheels, see Section 3.4- Step 1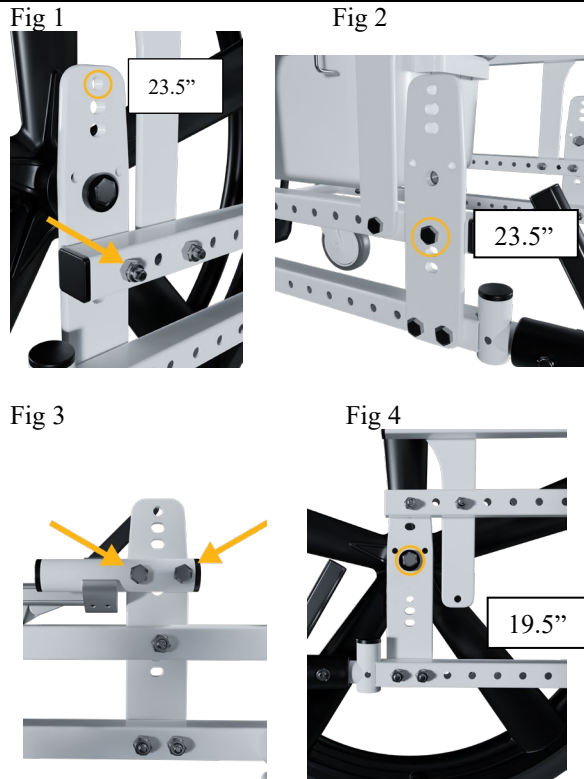

3.2 Back Height and Angle

The back height can be adjusted to 16", 18" or 20". To adjust the back height, follow these steps using ½" wrenches:

- 1) Remove two hex bolts on both sides of the back as shown in Fig 1.
- 2) Place back at one of three mounting locations. The location of the top plate for each height is noted in Fig 2.
- 3) Re-insert hex bolts and tighten securely.

Back Angle Adjustment

The back angle comes standard at 10°, but can also be adjusted to 5° or 15°. To adjust, remove the hex bolt from the top plate on both sides of the back, and re-locate the bolt to one of the other 2 holes as shown in Fig 3.

Fig 1

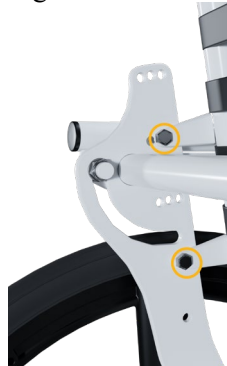
Fig 2

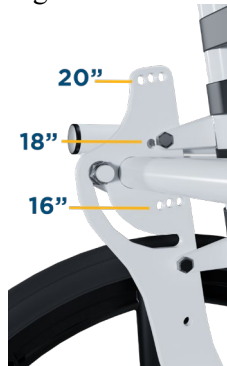
Fig 3

3.3 Seat Depth

The seat depth can be adjusted from 16"-19" in 1" increments. To adjust the seat depth, follow these steps using ½" wrenches:

1. Remove the two hex bolts on each side of the back as shown in Fig 1
2. Slide the back to the desired depth as shown in Fig 2. It is recommended that two people complete this step.
3. Re-insert hex bolts on each side and tighten securely.

Fig 1


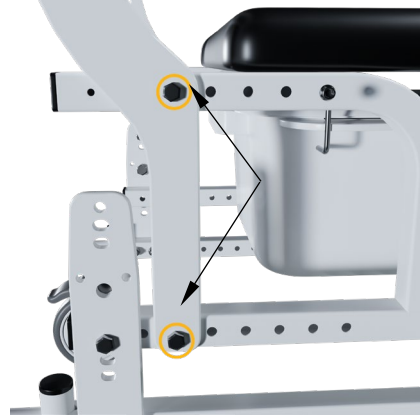
Fig 2

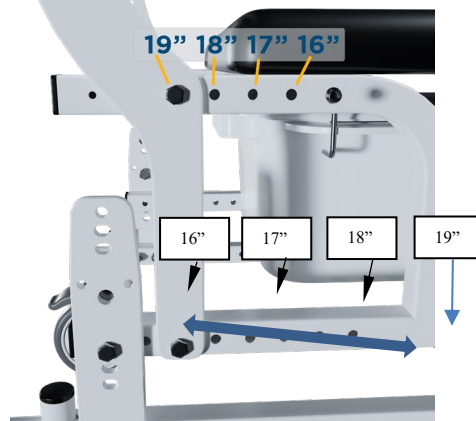3.4 Caster Conversion

The Sentinel comes standard with 24" mag wheels, however, it can easily be converted to an all caster wheelchair. Two 5" total lock casters will need to be purchased. To convert, follow these steps:

1. Remove the mag wheels by removing the axle bolt using a 5/8" wrench and a 5/8" socket, as shown in Fig 1.
2. Remove the two wheel locks by removing the hex bolts as instructed in Section 3.1- step 3
3. Remove the two anti-tippers by removing the screw as shown in Fig 2. You will need a 4mm Allen key and 10mm wrench.
4. Insert 5" casters into holes shown in Fig 3, lining up the hole in the frame with the hole of the caster tube

Fig 1

5. Insert screw provided and secure with acorn nut using a Phillips screwdriver and a 7/16" wrench, as shown in Fig 4.
6. If converting from all caster model to mag wheels, a mag wheel conversion kit will need to be purchased (MAG-CONV-CS313). Follow the above steps in the reverse order, starting with removing the casters. REF drawing X17101 on page 27 of the manual.

Fig 2

Fig 3

Fig 4


3.5 Mag Wheel Mounting Position

The mag wheel position can be adjusted forward or rearward. In the rearward position, this provides optimal clearance for lateral transfers (optimized when combined with 23.5" seat height). Adjusting the wheel further forward provides easier self-propelling (optimized when combined with 19.5" seat height).

There is 7" (14"-21") of range to find the optimal Rear Plate Position (RPP). To adjust the RPP, follow these steps using a 1/2" wrench:

1. Remove the 3 hex bolts on each side as shown in Fig 1.
2. Move the rear plate to the desired position shown in Fig 2.
3. Re-insert hex bolts into frame and tighten rear plate securely.
4. Adjust location of wheel locks by loosening 2 screws with a 5mm Allen key as shown in Fig 3.
5. Slide the wheel lock stem (Fig 5) forward or rearward, depending on new location of mag wheel, and re-tighten the screws securely. Pushing the handle should create a firm lock.

**Note- in RPPs 14"-17" the wheel lock mounting tube (Fig 5) needs to be re-oriented to point forward (flip 180°) to provide necessary clearance to lock in place

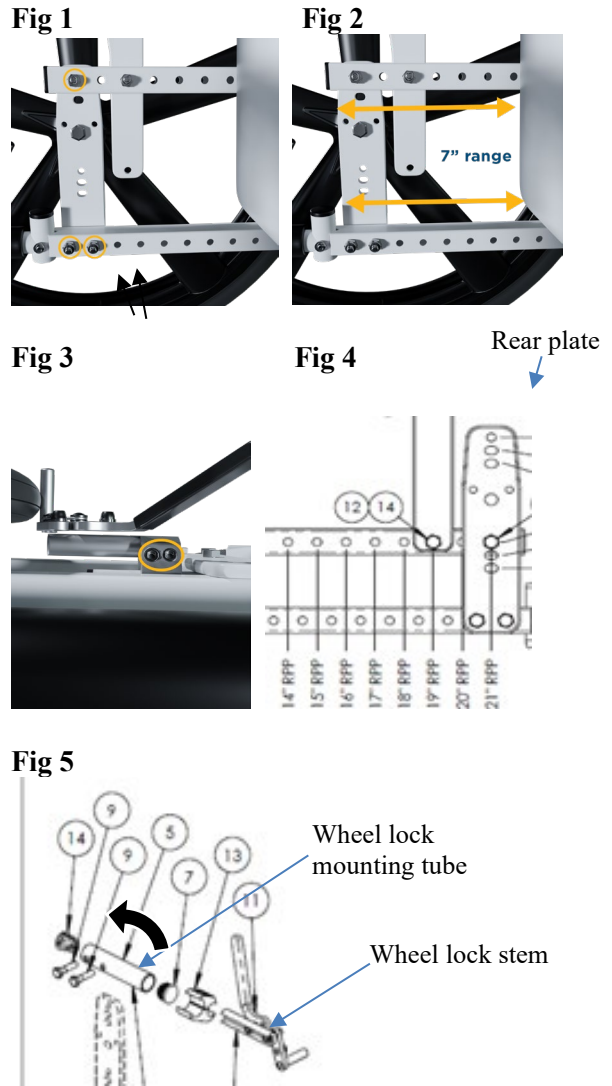3.6 Anti-Tipper Height Adjustment

The height of the anti-tippers can be easily adjusted and does not require any tools. Simply squeeze the round metal plate towards the rear as shown in Figs 1 and 2 and slide the bar vertically to desired height. (Do NOT remove adjustment stroke limiting fastener from anti-tipper adjustment tube)

3.7 Commode Seat Mounting

The commode seat has 4 positions with the gap facing forward, left/right, or rear. No tools are required to rotate the seat. Simply lift the commode seat off the seat rail, rotate and re-position by snapping clips back onto the rail.

Fig 1

Fig 2

Fig 3

3.8 Casters & Mag Wheels

The Sentinel features 5" stainless total lock casters and one (optional) rear 5" stainless rear directional locking (tracking) caster. The tracking feature eliminates any swivel function, which is ideal for pushing the chair down hallways.


The swivel casters are ideal for maneuvering in tight spaces and moving the chair laterally.

To Operate the Casters:

Step on the rear of the caster pedal to lock and on the front of the pedal to unlock the caster.

When using rear mag wheels, lock the wheels by pushing the lever to engage the brake on the mag wheel. Pull on the lever to disengage the brake on the mag wheel.

Always ensure the brakes are unlocked when wheeling the chair.

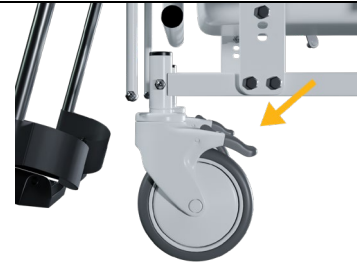3.9 Angle Adjustable Seat Dump

Adding angle to the seat can provide additional comfort and stability. The angle of the seat can be adjusted from 0° (standard) up to 6°.


To adjust the seat angle, follow these steps using ½" wrenches:

1. Remove the hex bolt on both sides of the rear mag plate as shown in Fig 1
2. Set plate to desired angle using holes indicated.
3. Re-insert hex bolts and tighten securely.

Fig 1

Rear mag plate

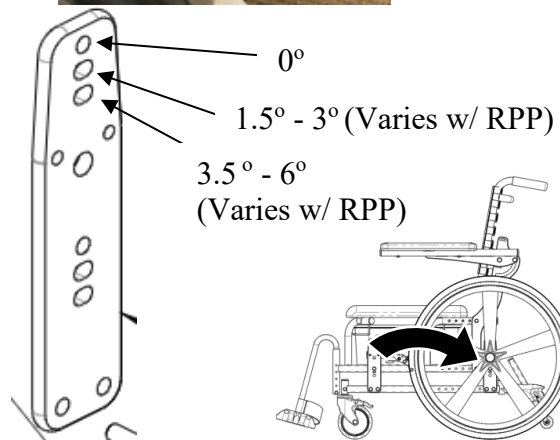3.10 Removing and Adjusting Footrests

The footrests can be easily removed by lifting the gooseneck bars upward, as shown in Fig 1. To swing away, lift slightly and rotate outwards, as shown in Fig 2.

To adjust the height of the footrests, follow these steps using a 7/16" wrench and Phillips screwdriver.

1. Remove the screw shown in Fig 3 and footrest
2. Insert and tighten screw with acorn nut into desired height (14"-18" knee-to-heel range) as shown in Fig 4 (Only tighten enough to seat the fastener. Do NOT over-tighten as this may permanently deform the tube and hinder gooseneck insertion.)
3. Re-insert gooseneck into hanger tube

Fig 1


Fig 2

Fig 3

Fig 4

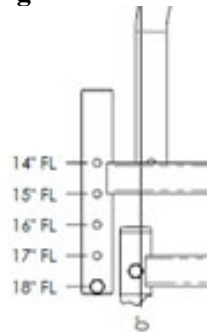
3.11 Removable Bed Pan & Basket (Option)

To remove the bed pan, lift the front lip of the pan slightly and then pull out from the rear (Fig 1). The wire basket can also be removed by pulling the 4 corners of the basket from the frame, as shown in Fig 2.

Fig 1


Fig 2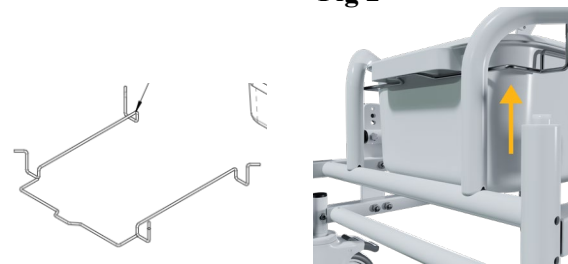

3.12 Lateral Torso Support (Option)

To attach the molded lateral supports, place the backing plate behind the back rest and line up the holes with the desired height of the pre-cut slots. Place the lateral support over the front of the back rest, lining up the holes with the backing plate. Insert the two thumbscrews from the rear and tighten securely.

Note: The width of the laterals can be easily adjusted by sliding the lateral support/plate horizontally in the back rest slots.

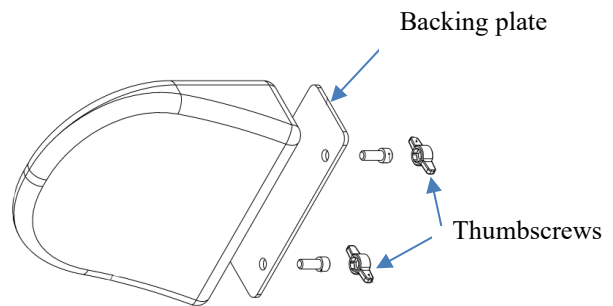3.13 Accessories (Options)

We recommend Broda accessories for use only on Broda chairs. The correct installation of the Broda accessory is very important. We recommend that the Broda chair is ordered with the required accessories already installed. Only authorized caregivers who thoroughly understand the accessory installation procedures should attempt to install their Broda accessories without assistance from their local Broda representative. Broda does not recommend use of other manufacturer's accessories on Broda chairs.

4 INSPECTION AND FUNCTIONAL TESTING

Safety measures as described in Section 1 must be observed when inspecting or testing a chair. Only authorized caregivers or maintenance staff should attempt to inspect or test a chair.

4.1 Inspection

We recommend regular visual inspection for signs of wear, damage, loose or missing fittings, and other safety concerns. If a breakage, defect, or operational problem is detected, the chair must be repaired inspected and tested for function before it is returned to service.

We recommend that the chair be inspected as often as each use if there is any reason to be concerned about the possibility of increased wear or loose or missing fittings. **At a minimum, in regular use the chair should be inspected on a bimonthly basis.** The visual inspection procedure should include at least the observation of all the fittings (fasteners):

The visual inspection procedure should include at least the observation of the following parts:

- 1) The vinyl backrest
- 2) The clips on the commode seat

The visual inspection procedure should include the observation of any installed accessories. The visual inspection should be performed by the facility, or if in a private residence, by the individual responsible for the chair.

4.2 Functional Testing

We recommend that the chair should be tested for operation of the chair's functions without a resident in the chair. The testing may be as often as each use if there is any reason to be concerned about the possibility of increased wear or damage to the chair's components. At a minimum, in regular use the chair should be tested for functions as described in Sections 3.

If the caregiver or maintenance department performing the functional testing believes that any function is not operating correctly, the chair should be taken out of service until a satisfactory functional test can be completed.

When performing repairs or maintenance, do not use lubricants that contain solvents. Solvents will damage many of the moving components in the chair. If necessary, a white, food grade grease (lubricant) may be used on the sliding components in the chair. Do not use spray lubricants on any part of the chair.

5 TECHNICAL INFORMATION

5.1 Specifications

Standard Features of Sentinel Shower Commode Chair

1. Frame: 16-gauge powder coated stainless steel
2. Chair Surface: Molded polyurethane foam seat/commode
3. Arms: Flip up
4. Footrest: Swing away and removable/adjustable footrest
5. Wheels: Two 5" front total lock stainless casters, two rear 24" mag wheels
6. Back Height: Adjustable 16", 18", or 20"
7. Back Material: Anti-microbial vinyl and tension adjustable straps
8. Seat Depth: 17"
9. Seat Height: Adjustable 19.5" or 23.5"
10. Seat Width: 18"
11. Overall Width: Seat width + 8" (w/ mag wheels), or seat width + 7" (w/ all casters)
12. Weight Capacity: 300 lbs.
13. Overall Weight: 65 lbs.

All dimensions are ± 0.5 ".

5.2 Broda Sentinel Shower Commode Parts Schematic Legend- 2022

Description	Part Number
Base Frame CS313 (width)	BASE-CS313
Caster 5" Stainless Steel Directional Lock	CASTER-5" DL-SS
Caster 5" Total Lock Stainless Steel	CASTER-5" TL-SS
Seat Frame CS313 (width)	SEAT-CS313
Footrest assembly CS313 (pair)	F-SW-CS313
Back Frame ADJ CS313 (width)	BACK ADJ-CS313
Back Frame CS313 (width)	BACK-CS313
Hand Grips CS313	GRIPS-CS313
Kit mag wheel CS313 24"	MAG-CONV-CS313
Armest Assembly CS313	ARM ASSY-CS313
Arm Pad CS313	ARM-PAD- CS313
Seat Pad CS313 (width)	SEAT-PD-CS313
Commode basket frame CS313 (width)	COMMODOE-BSKT-CS313
Commode pan with lid CS313	BED-PAN-CS313
Commode seat clips CS313	SEAT-CLIPS-CS313
Back rest pad with slots (width)	BACK PAD-SLOTS-CS313
Back rest pad without slots (width)	BACK PAD-CS313
Anti-tipper assy CS313 (pair)	ATIP-CS313
Wheel lock assembly stainless steel (pair)	WC-BRAKE-SS

6 TROUBLESHOOTING AND REPAIR

Please have the chair serial number ready when contacting Broda or your local area representative for parts or assistance for your chair. The serial number can be found on a grey sticker on the rear of the chair.

7 MANUFACTURER WARRANTY

Effective 10/15/2018

From date of purchase Broda warrants the side frames and cross members located on the base frame, for the Expected Lifetime* of the original purchaser/user, when purchased from an Authorized Broda Dealer. *Expected Lifetime of the frame is 10 years.

In addition, Broda warrants its other product components, except the seat cushion and covers (which are not warranted), to be free from defects in materials and workmanship for a period of 2 years from the date of purchase, as indicated on the original purchase order. This warranty is non-transferable.

The warranty is subject to the following conditions:

A serial number or invoice number is required for warranty coverage. The warranty shall not apply to serial numbered items if the serial number has been removed or altered in any way.

Warranty is valid for “new” purchases only. All other items will be warranted at the sole discretion of Broda.

If within such warranty period any such product shall be proven to be defective through examination, inspection and testing as deemed necessary, such product shall be repaired or replaced at Broda’s discretion. This warranty does not include on-site labor.

On request, defective parts must be returned to the factory, pre-paid and insured, within 30 days. Failure to do so, will result in the owner incurring the cost of the replacement parts. Photos of defective parts may be requested as an alternate to returning product.

Limitations and Exclusions:

This warranty does not cover damages that arise from improper handling, cleaning, maintenance, storage or negligent use. The warranty does not cover Broda’s wheelchairs and chairs used in any way, other than the manner in which it was designed and recommended.

Modifications, incorrect assembly or installation to Broda products or the use of non-Broda products voids the warranty. Installation of parts that require the frame to be altered in any way, must be completed by a Broda Representative, unauthorized performances will void the warranty.

This warranty does not cover freight damage (see Damaged Freight Policy).

Broda retains the right to make product design and product application changes without notice.

The application of Broda products shall remain the responsibility of the purchaser or user. Please reference the model specific operating instruction manuals for safety requirements, inspectional and functional testing, technical information and more.

Warranty does not include on-site labor for the installation of warranty parts or warranty repairs. The owner may return to Broda products for warranty replacement or repair by shipping items prepaid and insured to the factory. Warranty completed at the factory includes both materials and labor. The decision to repair or replace parts is at the discretion of Broda. All returns to the factory require prior authorization from Broda.

WC-19 Transportation Certified Products:

Vehicle Transport Products and Options are available by factory install only. It can NOT be retrofit to existing models or serviced in the field. All components of the vehicle transport packages are subject to specific maintenance requirements, to maintain the Broda warranty.

Due to the testing requirements of the vehicle transport chair, Broda warranty limitations apply to non-factory installed repairs. Warranty and maintenance repairs must be made by Broda authorized service technicians. Any replacements or repairs made by unauthorized person (s) will void this warranty and Broda assumes no liability on the performance of said components.

DO NOT modify the transport chair. Modifications may change the transport chair’s structure and center of balance, which may increase the risk of serious injury to the user and void the warranty. If a problem is detected with any of the Vehicle Transport Product and Option components or any components of the chair, remove the wheelchair or chair from service and contact Broda immediately. Chairs involved in vehicle accidents should be taken out of use immediately.

For any further questions regarding our Manufacturer Warranty please contact Broda Customer Service at 1-844-552-7632 or customerservice@Brodaseating.com

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