Operating Manual

Traversa Transport Wheelchair

Distributor/Rep:

This manual must be given to the caregiver responsible for this chair and its occupant.

Caregiver:

Before using this chair, read this manual thoroughly and save for future reference.

Broda

15455 Conway Road Suite 200
Chesterfield, Missouri USA 63017

Phone: 1-844-552-7632

Fax: 844-442-7632

sales@brodaseating.com

brodaseating.com

OM-Traversa-4.3.2024

Actual product may differ slightly from images in Operating Manual

Contents

FOREWORD	1
GENERAL INFORMATION	2
1 DEFINITIONS	3
2 SAFETY REQUIREMENTS	4
2.1 Training.....	4
2.2 Application.....	4
2.3 Shipping and Storage.....	5
2.4 Pre-Service Check.....	5
2.5 Hazards.....	5
2.5.1 Position of Chair - “Danger of Falling”.....	5
2.5.2 Position of Seat Tilt - “Danger of Tipping”.....	5
2.5.3 Location of Chair - “Danger of Tipping or Falling Objects”.....	6
2.5.4 Total Lock Wheel Brakes - “Danger of Falling”.....	6
2.5.5 Re-Positioning of Resident - “Danger of Clamping”.....	6
2.5.6 Unintended Movement - “Danger of Falling or Collision”.....	7
2.5.7 Occupant Clothing - “Risk of Injury to Resident’s Skin”.....	7
2.5.8 Improper Restraint Use - “Risk of Serious Injury”.....	7
2.6 Improper Use.....	8
2.7 Cleaning.....	8
2.8 Maintenance.....	10
2.9 Occupant Specific Instructions.....	11
3 OPERATION AND MOVEMENT	12
3.1 Seat Tilt.....	12
3.2 Back Recline.....	12
3.3 Trendelenburg.....	13
3.4 Seat Height.....	13
3.5 Elevating Leg Rest.....	14
3.6 Wheel Locks and Brakes.....	14
3.7 Removable Wings.....	15
3.8 Shoulder Bolsters.....	15
3.9 Adjustable and Removable Arms.....	16
3.10 Height Adjustable Armrests.....	16
3.11 Anti-Tippers.....	17
3.12 Pivot Assist™.....	17
3.13 IV Pole and O2 Holder.....	18
3.13 IV Pole and O2 Holder.....	19
3.14 Motor Vehicle Use.....	20
Instructions.....	21
WARNINGS!	21
4 INSPECTION AND FUNCTIONAL TESTING	22
4.1 Inspection.....	22
4.2 Functional Testing.....	22
5 TECHNICAL INFORMATION	23
5.1 Specifications.....	23
6 COMMON TROUBLESHOOTING AND REPAIR	24
6.1 Troubleshooting for Gas Cylinders.....	24
6.2 Adjustment Steps.....	25
MANUFACTURER WARRANTY	26
MANUFACTURER WARRANTY	27

FOREWORD

Congratulations on your purchase of a Traversa by Broda chair and thank you for your confidence in our company and products.

Broda assumes a leadership role in providing optimum re-positioning functions and mobility for residents of long-term care institutions. Our chairs do not look like traditional chairs and offer advantages unique to Traversa by Broda. We are sure that after using your chair, you will be convinced that your resident's quality of life will be greatly enhanced.

This manual will assist you in making the best use of the capabilities of your Traversa by Broda chair and will ensure that you quickly become familiar with its operation.

After reading this manual, if you have any questions about the safe and effective operation of your Traversa by Broda chair or accessories contact your local Broda representative or our Head Office Customer Service personnel for further assistance.

GENERAL INFORMATION

This document provides guidance on the safe and effective operation of the Traversa by Broda chair.

Information in this manual must be followed at all times.

Anyone involved with the operation or maintenance of the Traversa by Broda chair, including the occupant's family members, and any vehicle operator must read this operating manual before using the chair.

The operator is responsible for ensuring that anyone who is unfamiliar with, unwilling, or unable to adhere to the safety and operating instructions, is not permitted to operate or move the chair.

A copy of this instruction manual must always be available.

Broda accepts no liability for damages, injury or accidents caused by operating errors, improper maintenance, or disregard of the instructions in this manual, including any resident specific instructions.

Broda reserves the right to make changes to the specifications, dimensions, functions, or components of its products without notice. Product representations in this manual may vary from delivered products.

Each Traversa by Broda chair has a unique identifying serial number that must be maintained on the chair as well as with any equipment records.

1 DEFINITIONS

Broda – *Broda Enterprises Inc. doing business as Broda Seating.*

Traversa – *Sub-brand of Broda Enterprises Inc.*

“Traversa by Broda chair” and “Broda chair” – *Chair model Traversa Transport Wheelchair.*

Long-Term Care Institution – *A nursing home, hospital, or other healthcare facility that provides health and personal care to its residents on a long-term basis.*

Resident or Occupant – *The individual occupying the chair and under care of professional caregivers.*

Professional Caregiver – *Doctors, nurses, therapists, nursing aids, healthcare aids, and other specialists who work in a long-term care institution and provide health and personal care to its residents.*

Caregiver or Operator – *The NEMT operator or any person in a long-term care institution who is appropriately trained to provide care or services to the occupant. This may also include the resident’s family members or guardian.*

Seat Tilt – *Changing the relative angle between the chair’s seat and the chair frame or ground without changing the relative angle between the back and the seat.*

Back Recline – *Changing the relative angle between the chair’s back and the seat.*

Footrest Elevation and Leg Rest Elevation – *Changing the relative angle between the chair’s footrest and/or leg rest and the seat.*

Transfer(s) – *Movement of an occupant into or out of the chair with the assistance of their caregiver(s).*

Mechanical Transfer(s) – *The movement of a resident into or out of the chair with the assistance of their caregiver(s) using a patient lift or other assistive device that bears the weight of the resident.*

2 SAFETY REQUIREMENTS

2.1 Training

Before the chair is put into service, this manual must be read thoroughly by the caregiver(s) directly responsible for the resident's care.

After the chair is put into service, this manual must be read thoroughly by any new caregivers prior to operating or moving the chair.

For the purpose of this manual, a resident's family member who shares responsibility for their care is considered a caregiver and is subject to the same competency before being permitted by the resident's primary caregiver to operate or move the chair.

Prior to first use, the customer must arrange for an in-service on the operation and safety requirements in this manual, must be given to the resident's caregivers by the local Traversa representative who supplied the chair.

The primary caregiver must maintain a list of caregivers who have read this manual and who they have authorized to operate and move the chair.

The resident's primary caregiver is responsible for ensuring that anyone who is unfamiliar with, unwilling, or unable to adhere to the safety and operating instructions, is not permitted to operate or move the chair.

The operations of the chair must be performed by the resident's primary caregiver who is responsible for seating. All The operations and adjustments performed should be done in a manner to ensure the overall safety, comfort and well-being of the resident, caregiver and third party. All operations and adjustments required for the resident should be determined by the resident's primary caregiver who is responsible for seating.

2.2 Application

Traversa by Broda chairs are intended for occupants in NEMT transportation vehicles under the care of professional operators. The suitability of a chair must be determined by a qualified operator who is familiar with the travel needs of the intended occupants. Any other use of the chair is excluded from possible liability claims.

The Traversa is not to be used in the shower. The frame and components will rust and will void the warranty.

The chairs are not explosion resistant and must not be used where there are flammable gases or liquids present (e.g., anesthetics, volatile solvents and cleaners, etc.).

Broda chairs are designed for use with specific Broda parts and accessories. The use of non-Broda parts or accessories with a Brodwwwwwwa chair will void the warranty and is excluded from possible liability claims.

Broda chairs may only be used as described in this manual and with proper regard for recognized healthcare and workplace safety and accident prevention practices.

2.3 Shipping and Storage

Broda chairs should be shipped and stored in an upright position and not stacked higher than 3 boxes. No other materials should be shipped or stored on top of a Broda box.

Broda chairs should be shipped and stored at temperatures between -20°C and 40°C. Broda chairs should not be used until they are between 0°C and 30°C.

Broda chairs should be kept in a clean, dry environment. Do not leave Broda chairs outdoors as it may cause the paint to peel.

Upon receipt, the shipping carton must be immediately examined for damage. Any damage should be noted on the delivery receipt and a request for inspection by the transportation company should be made. Next, the shipping carton should be opened and the chair must be examined for concealed shipping damage. If the chair appears to be damaged, do not use the chair. File a concealed damage report with the transportation company.

2.4 Pre-Service Check

Broda chairs are delivered fully assembled. If the chair does not appear to be ready to use upon receipt, immediately contact your supplier and do not put the chair into service until any concerns have been resolved.

Visually inspect the chair for damage, missing parts, and loose fasteners prior to testing the chair's functions. Functional testing must be successfully completed after visual inspection and before use. These obligations apply to the chair's first use and to all subsequent uses (Section 4: Inspection and Functional Testing).

2.5 Hazards

2.5.1 Position of Chair - "Danger of Falling"

After a resident is transferred into a chair, assess the amount of tilt required. We recommend that the chair's seat be tilted sufficiently to prevent the resident from sliding or falling forward off the chair. The amount of seat tilt used should be determined by the resident's caregiver who is responsible for seating.

We recommend that the resident's feet be correctly positioned on the footrests and slightly-to-fully elevated to prevent the resident from sliding or falling forward off the chair. The amount of elevation used should be determined by the resident's caregiver who is responsible for seating.

2.5.2 Position of Seat Tilt - "Danger of Tipping"

We recommend that the chair's seat be tilted sufficiently to prevent an agitated resident from tipping the chair forward or backward, or from slumping and sliding in the chair. The amount of seat tilt used should be determined by the resident's caregiver who is responsible for seating. Always ensure that the resident is properly positioned before operating the seat tilt.

2.5.3 Location of Chair - “Danger of Tipping or Falling Objects”

We recommend that when a resident has been moved to their destination, the chair is placed where the resident cannot reach handrails or other objects, fixed or movable. This is to prevent the resident from pulling the chair over or pulling themselves off the seating surface and to prevent the resident from pulling movable objects onto the chair and themselves.

We recommend that the chair be used in a supervised area to prevent untrained residents, caregivers, or third parties from unauthorized operation, movement, or unsafe actions such as sitting or leaning on the reclined back, elevated footrest, or the armrests. These actions, if not prevented, put the chair at risk of tipping or damage to the chair.

We recommend that a chair only be located on a level surface to minimize the risk of tipping over.

2.5.4 Total Lock Wheel Brakes - “Danger of Falling”

The wheels found on the Broda chair have total lock brakes which prevent the wheels from turning and swiveling. The brakes must always be applied when:

- 1) The chair is not in use;
- 2) A resident is being transferred (moved) into or out of the chair; and,
- 3) The chair is not being moved by a caregiver.

It is important to note that if the wheel locks are applied while the patient is in the chair, that the caregiver does not leave the patient unattended, especially those patients who have the capability or tendency to move the chair and/or those who may be agitated. This could cause harm to the patient if they attempt to move the chair while the wheel locks are applied.

Note that removing and attaching the footrest may be easier for the caregiver with the brakes applied.

Failure to follow these instructions will unnecessarily increase the risk of serious falls by residents, caregivers, or third parties caused by the chair unintentionally moving.

2.5.5 Re-Positioning of Resident - “Danger of Clamping”

Broda chairs offer the benefits of seat tilt, back recline, leg rest/footrest elevation, and moveable arms. During the movement of any of these functions, the following safety measures must be observed:

- 1) The occupant’s arms must be positioned on the armrests or inside the chair frame with their hands on their body.
- 2) The occupant’s feet must be correctly positioned on the footrest.
- 3) All of the chair’s brakes have been applied.
- 4) Only one caregiver at a time attempts to operate the chair’s functions.
- 5) Only one chair function is operated at a time.
- 6) The rear wheels are in the trailing position, behind the chair frame.
- 7) The residents’ and caregivers’ body are clear of all pinch points before operating the chair’s functions.

Failure to follow these safety measures can put the occupants' or caregivers' limbs at risk of injury. Occupants who may be unaware of their body position or unable to maintain a safe body position are at the most risk of the danger from clamping and caregivers should be more cautious with these occupants. A second caregiver may be required to ensure the safety of these occupants during these operations.

2.5.6 Unintended Movement - "Danger of Falling or Collision"

We recommend Broda chairs for use with NEMT vehicles and where there is not enough slope to cause the chairs to move unaided. Chairs used where the surface is uneven or sloped are at risk of unintended movement and could become a serious danger to the resident, caregiver(s), or a third party. We recommend that Broda chairs are located away from stairwells, elevators, and exterior doorways within a long-term care institution.

Outdoor use is appropriate only under the strict supervision and full attention of a caregiver who is physically capable of preventing unintended movement over any surfaces that are to be traveled on.

2.5.7 Occupant Clothing - "Risk of Injury to Resident's Skin"

We recommend that occupants only be seated while they are fully dressed in clothing that meets the needs of their specific condition. If after being fully dressed, an occupant's bare arms, legs, or body could still come into direct contact with the vinyl pads, we recommend a folded cloth bed sheet to prevent direct contact. Prior to the operation or movement of a chair with a covering, the caregiver must ensure any covering placed on the chair does not come into contact with any moving parts and is securely placed, so not to slip.

2.5.8 Improper Restraint Use - "Risk of Serious Injury"

We recommend that alternatives to physical restraints be used with occupants while seated in the chair except under the specific instructions of the occupant's primary caregiver and with permission of the occupant's family or guardian. Physical restraints have been identified as a common cause of serious injury to residents while they are seated. We recommend that the primary caregiver responsible for seating first consider the re-positioning options available in the chair to reduce the risks of sliding, falling, or self-injury.

If a physical restraint is determined to be appropriate to prevent sliding or falling out of the chair, we recommend that the Broda thigh belt be used.

2.6 Improper Use

As outlined, the improper use of the chair is dangerous to the occupant, caregivers, or third parties, and can consist of, but is not limited to the following:

- 1) Unauthorized operation of the chair's functions.
- 2) Unauthorized movement of the chair.
- 3) Inappropriate use of the chair for an occupant who has not been assessed by a qualified caregiver responsible for their seating.
- 4) Failure to frequently reposition the resident in the chair.
- 5) Attempting to operate multiple chair functions simultaneously by one or more caregivers.
- 6) Attempting to move the chair with the brake(s) applied.
- 7) Leaving the occupant unattended in the chair near other objects.
- 8) Leaving an agitated occupant in the chair in an unsupervised area.
- 9) Leaving an occupant unattended.
- 10) Leaving an occupant in a chair on a sloping surface.
- 11) Leaving a chair unattended on a sloping surface.
- 12) Using non-Broda accessories on the chair.
- 13) Using the chair at temperatures below 0°C.
- 14) Using the chair as a shower or bathing chair.
- 15) Using the chair for any use other than its intended purpose.

2.7 Cleaning

Broda chairs should be wiped clean with soap and water. Diluted household strength ammonia or chlorine based cleaner, as well as a hospital grade cleaner may be used if necessary.

Refer to the following guide for cleaning certain staining agents (Applies to the Permablok3® vinyl, as per the vinyl manufacturer's recommendations). The frame and components of the chair can be cleaned using the same procedure as the vinyl.

The steps below are located in the Cleaning Guide on the back of the Broda chair.

Staining Agent	Cleaning Procedure
Eye Shadow / Mascara	STEP 1
Grease / Suntan Lotion / Chocolate	STEP 2
Blood / Bodily Fluids	STEP 3
Red Lipstick	STEP 4
Oil Base Paint / Tar / Asphalt	STEP 5
Other Tough Stains	STEP 4 OR 5*

STEP 1: Remove excess spill with a damp cloth. Clean with a 1:1 mix of Ivory® liquid and water. Rinse with clean water and dry.

STEP 2: Use a straight application of concentrated cleaners such as Formula 409® or Fantastik® Spray Cleaner. Then wipe with a clean cloth. If using hospital grade cleaner, follow diluting instructions on the label.

STEP 3: Use a 1:1 mix of ammonia and water or a 1:4 mix of bleach and water. Rinse with clean water and dry.

STEP 4: Use a straight application of naphtha (lighter fluid). Rinse thoroughly with clean water and pat surface dry.

STEP 5: Use a 1:1 mix of isopropyl alcohol and water. If the stain persists, use straight alcohol. Rinse thoroughly with clean water and pat dry. If the stain remains, use a 1:1 mix of acetone and water. Rinse with clean water and pat surface dry.

***NOTE:** For cleaning that requires steps 4 or 5 – Use a soft cotton cloth saturated with the cleaning material and rub the stain in circles 10 times. Pat dry with another soft cotton cloth, and check results.

For best cleaning results, certain parts and padding on the chair should be removed: Instructions for removal of the following parts are located in this manual: Ensure that the entire chair including removable and non-removable parts is thoroughly cleaned.

Pay close attention to the frame, sockets, fasteners, parts and casters, as well as the padding for an even, thorough clean.

Remove parts for cleaning if desired:

- 1) Lateral Shoulder Supports
- 2) Removable Arms
- 3) Footrest
- 4) Padding

Thoroughly clean all removable parts and padding, as well as the frame and components around the parts and padding that have been removed.

Broda chairs should not be cleaned with petroleum-based cleaners. Any petroleum based products that come in contact with any vinyl surface should be removed quickly as possible. Petroleum based products make vinyl brittle and will damage seating surface and cushions. Metal parts and cushions should be wiped completely dry after cleaning. Do not launder vinyl padding. Terry cloth seat and back covers can be laundered and tumble dried.

Do not allow the chair to air dry. Leaving the chair to air dry and not ensuring the entire chair is completely dry may cause the frame and components to rust. Solvents such as those found in spray lubricants should not be used on Broda Chairs as they can damage moving parts.

Frequency and method of cleaning the chairs should be determined by facility infection control protocols. If visibly soiled, thoroughly clean the chair immediately as per the cleaning instructions in this manual.

The cleaning instructions in this manual are guidelines only. Results may vary under actual conditions. The information does not relieve the user of proper and safe use of the product and all cleaning agents and consideration for the overall cleaning maintenance of the chair. Cleaning and care instructions must be followed in conjunction with facility infection control protocols.

Broda will not accept warranty or liability claims on chairs that have not been cleaned according to the instructions or cared for in proper regard for patient, caregiver and third-party safety and hygiene. The use of certain agents may be harmful to the surface appearance and lifespan of the vinyl. Broda assumes no responsibility resulting from the use of such cleaning agents to the vinyl.

Touch up paint is available for the powder coated frame. Please call Broda's Customer Service Department if touch up paint is required. Do not leave Broda chairs outdoors as the frame is not UV protected. Leaving the chairs outdoors could cause the paint to peel.

Formula 409® is a trademark of the Clorox Company.

Fantastik™® is a trademark of DowBrands, Inc.

Trademark of the Dow Chemical Company.

Ivory® is a trademark of Proctor and Gamble.

2.8 Maintenance

The maintenance on a Broda chair will vary with the amount of use and the condition of the occupant using the chair.

In regular use, after the initial inspection and functional testing, the chair should be inspected and tested bimonthly. We recommend visually inspecting for signs of wear, damage, loose or missing fasteners, and other safety concerns. Periodic testing of chair functions is also necessary. If a breakage, defect, or operational problem is detected, the chair must be successfully repaired, inspected and tested for function before it is returned to service.

The chair should be inspected and tested as often as each use if the chair is used by:

- 1) Aggressive or agitated occupants.
- 2) Occupants who have involuntary movements.
- 3) Occupants weighing over 220 lbs.
- 4) A facility with irregular or sloped surfaces.
- 5) Any unauthorized person.

Do not use lubricants that contain solvents. Solvents may compromise any rubber components on the chair. If necessary, white food grade grease or lubricant can be used on any sliding components.

2.9 Occupant Specific Instructions

The primary caregiver responsible for the specific occupant's seating shall add additional instructions necessary for the safe and effective use of the chair based on their professional experience and knowledge of the occupant's specific conditions and requirements.

Broda representatives are not professional caregivers and will not know the specific requirements of the individual using the chair. Broda relies on the knowledge, experience, and judgment of the occupant's professional caregiver to ensure the specific occupant's safety and comfort needs are satisfied while using the chair.

These instructions form an essential part of the Safety Requirements for using the chair and must be made available to all caregivers. Space is provided below to include these instructions.

Broda
15455 Conway Road Suite 200
Chesterfield, Missouri USA 63017
Téléphone: 1-844-552-7632
Fax: 844-442-7632
sales@brodaseating.com
brodaseating.com

3 OPERATION AND MOVEMENT

Safety measures as described in Section 1 must be observed when operating the chair's functions or moving the chair. Only authorized caregivers should attempt to operate or move a chair. The caregiver must use proper body mechanics when operating the chair, and be prepared to support the weight of the passenger while operating the chair.

3.1 Seat Tilt

Using the seat tilt feature helps keep passengers comfortable, especially on long trips. Specially pressurized gas cylinders allow the seat to be tilted to any position from most upright to the lowest within its range.

Use seat tilt to safely ease your passenger into a comfortable position. Placing your Traversa wheelchair in tilt also lowers the height of the wheelchair, making it easier to fit through vehicle doors or other areas of low clearance.

To Activate the Seat Tilt:

- Squeeze tilt handle with one hand while placing your other hand on the bar to assist movement.
- Raise or lower the seat tilt until the desired.

3.2 Back Recline

Your Traversa Transport Wheelchair features a reclining back. The wheelchair can recline from upright to a flat position or anywhere in between. Use back recline to position your client optimally for a lateral transfers or simply to increase their comfort. Always be sure to tilt the wheelchair slightly prior to reclining. This prevents the occupant's back from sliding along the back rest during recline, which can cause damage to the skin. Always bring the occupant back to an upright seated position after they have boarded your vehicle. Passengers should never travel in a reclined position.

To Activate the Back Recline:

- Squeeze recline handle with one hand while placing your other hand on the bar to assist movement.
- Raise or lower the back until the desired angle is achieved.

3.3 Trendelenburg

The Trendelenburg feature allows caregivers to quickly position the feet above the head.

To Activate Trendelenburg:

- Recline fully, then tilt fully.

3.4 Seat Height

Before raising or lowering the wheelchair, make sure you have engaged the wheel locks.


To lock the rear wheels in place:

- Activate the rear foot pedal by applying force to the pad closest to the back of the wheelchair.
- To release the lock, apply force to the pad closest to the front of the wheelchair.

The seat of your Traversa Transport Wheelchair raises from 22.5 inches to 32 inches high. This can assist you in performing lateral transfers.

To raise/lower your chair:

- Begin by placing the wheelchair into the fully reclined position.
- To raise the seat height, place one foot on the foot pump and press downwards repeatedly on the pad closest to the rear of the wheelchair. Each pump will raise the chair approximately ½ inch.
- To lower the seat height, use your foot to press down and hold the pad closest to the front of the wheelchair. As you hold, the wheelchair will gradually lower to its original position. jusqu'à sa position d'origine.

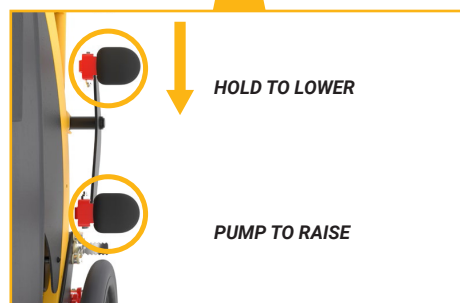3.5 Elevating Leg Rest

The leg support on the Traversa can be raised or lowered to make your passenger comfortable. Gas cylinders allow you to raise the leg rest from the lowest to the highest position, or anywhere in between. The leg rest elevates and extends automatically by recline feature on the Traversa, but you can adjust the height independently. If the back is in reclined position, the leg rest must stay extended and cannot be lowered.

To Adjust the Leg Support:

- Hold leg support and squeeze the leg rest handle. ●
- Raise or lower the leg rest to the desired angle. ○

To adjust the length of the footrest, squeeze the footrest handle and pull on the footrest to desired location.

3.6 Wheel Locks and Brakes

To lock the rear wheels in place:

- Activate the rear foot pedal by applying force to the pad closest to the back of the wheelchair.
- To release the lock, apply force to the pad closest to the front of the wheelchair.

The Traversa comes equipped with a handactivated brake for stability and control while going down ramps and graded surfaces.

To slow or stop the wheelchair while in motion:

- Squeeze the hand brake, as you would a bicycle brake. To release the brake, simply let go of the handle.

WARNING: *The brake is only engaged when continually squeezing the handle. To prevent the chair from rolling and safely secure occupant, activate wheel locks.*

3.7 Removable Wings

Wings support the Shoulder Bolsters which provide upper body lateral support. The wings are width and height adjustable to accommodate various occupants.

To Remove the Wing:


- Loosen the knob (1) and pull the wing from the receptacle.

To Insert the Wing:

- Insert the wing back into the receptacle.
- Tighten the knob and ensure the wing is secured in place.

To Adjust the Wing Height:

- Loosen the knob and raise or lower the wing to the desired position.
- Tighten the knob and ensure the wing is secured in place.

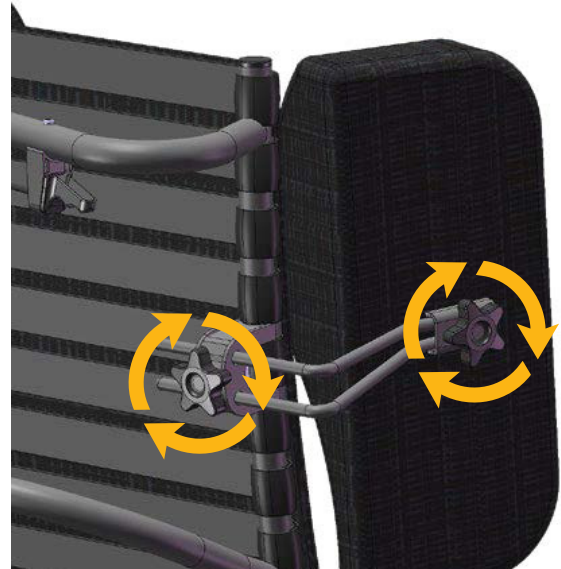3.8 Shoulder Bolsters

The Shoulder Bolsters provide upper torso support, protection and containment and accommodate asymmetries in the chair user. Their unique design allows for proper elbow access to the arm rest. The Shoulder Bolster is always used in conjunction with the side wing.

To Attach the Shoulder Bolsters:

- Fit the Shoulder Bolster over the wing.
- Secure with the Velcro strips.

Note: *The Shoulder Bolster can be inverted on the wing in order to provide support to narrower occupants.*

3.9 Adjustable and Removable Arms

The arms on the chair are width adjustable to allow for up to 4" (2" on either side) of additional width for the occupant. They are also removable to allow for side access to facilitate safe and easy transfers.

To Remove the Arms:

- Release both red handles and slide the arms out until rods are completely out of the sockets.
- If additional width is desired, pull the arms to the desired location, but do not exceed the groove, which is the 2" point.

To return, insert the rods into the sockets, ensuring the handles are facing down, with the warning labels facing the wheelchair. Lock the handles in place.

3.10 Height Adjustable Armrests

The armrest height can easily be adjusted in order to accommodate various occupants. The armrest height should be adjusted to provide correct arm support for the occupant as determined by the operator.

To Adjust the Armrest Height:

- Squeeze the pins together which are located under the armrest flap.

WARNING: DO NOT place fingers at the top of the pins as this could cause pinching. Place fingers in the concave shape of the pins.

- Raise or lower the arm to the desired height.

WARNING: DO NOT place fingers anywhere underneath the arm while adjusting, as this could cause pinching. Upon releasing the pins the armrest height will lock into place.

Secure the outside flap on the side panel. Change the height on the other arm as desired.

3.11 Anti-Tippers

The rear anti-tip devices can be adjusted easily, which can provide clearance when going up steep ramps, curbs, hills, or other obstacles.

To adjust the height of the anti-tip devices, press and hold the red button and raise or lower the antitip device to release it from current setting.

- When adjusting, the anti-tip device will lock into the next setting. There are three pre-set heights to choose from.

WARNING: For optimal safety, the anti-tip devices should be in the lowest position at all times when an occupant is in the wheelchair. Only raise the height if necessary, and take extra precautions as the wheelchair is at greater risk of tipping.

3.12 Pivot Assist™ - Accessoires (Options)

For tight spaces, the Traversa Pivot Assist™ System allows the operator to swivel the chair a full 360° without having to lift the wheelchair.



To activate the Pivot Assist™ System:

- Flip down the foot pedal.
- Squeeze handle on side of bar.
- While squeezing the handle, press down on the foot pedal where the rear of the wheelchair lifts off the ground.
- Once foot pedal is depressed, release the handle.
- Rotate wheelchair as required.

To de-activate the Pivot Assist™ System:

- Squeeze handle on side of bar, dropping the wheelchair back down.

Note - This feature is factory installed only and cannot be added after-market. It is highly recommended this feature be added should operators encounter any small spaces where rotating the wheelchair is difficult to do.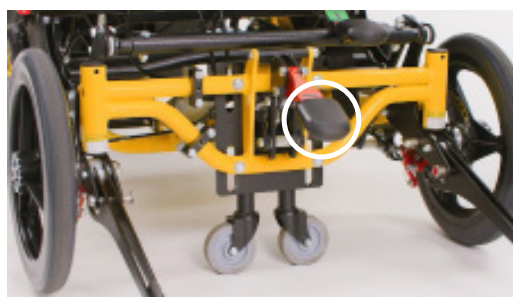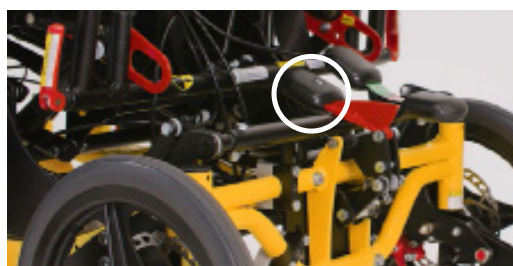

3.13 IV Pole and O2 Holder

The IV pole comes installed when ordered as part of the Traversa.

To move the location to the opposite side:

- Remove the 3 bolts with a ½" wrench (see pic).
- Remove the IV pole holder and re-insert the bolts.

Remove the 3 bolts on the opposite side, insert the IV pole holder, and secure with same 3 bolts.

To remove the IV pole only:

- Open red lever as shown so it's facing down.
- Remove the pole by lifting up.
- When re-inserting the IV pole, secure by closing red lever until tight.

The O2 Holder comes installed when ordered as part of the Traversa.

To adjust the height of the holder, loosen the bolts using a ½" wrench (see pic), adjust the height of holder, and re-tighten bolts.


To adjust the angle of the holder, loosen the two bolts in the top clamp using a 7/16" wrench and slide the clamp to the right across the frame (see pic). Re-tighten bolts.

To angle the opposite way, loosen the two bolts in the lower clamp, slide the clamp to the right across the frame, and re-tighten bolts.

Note - Angling the O2 holder will provide additional clearance for tilt and recline.

3.13 IV Pole and O2 Holder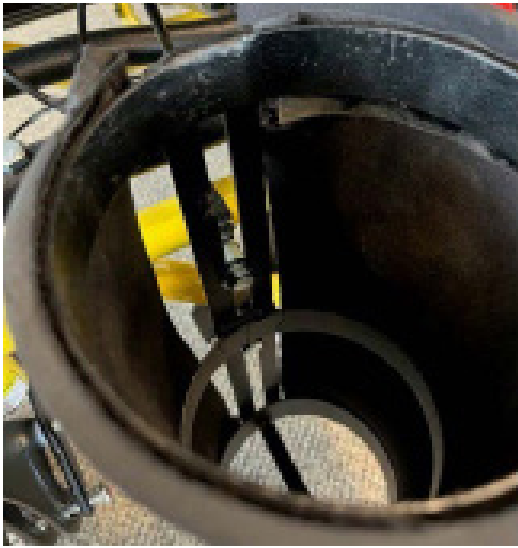

**Transfer Rollboard with
holder also available!**

We recommend Broda accessories for use only on Broda chairs. The correct installation of the Broda accessory is very important. We recommend that the Broda chair is ordered with the required accessories already installed. Only authorized caregivers who thoroughly understand the accessory installation procedures should attempt to install their Broda accessories without assistance from their local Broda representative. Broda does not recommend the use of other manufacturers' accessories.

3.14 Motor Vehicle Use

This chair is equipped for safe vehicle transport.

The Traversa Transport Wheelchair conforms with requirements of ANSI/RESNA WC/Vol. 1- Section 19 vehicle crash test standards with a 95% percentile large male ATD.

Testing was conducted in accordance with procedures set forth in Annex A of Section 19, ANSI/RESNA WC/Vol. 1 Wheelchairs Used as Seats in Motor Vehicle, hereafter referred to as ANSI/RESNA WC19 or WC 19. The chair's performance has been measured and evaluated according to the requirements of 5.3 of this standard. The wheelchair ATD was restrained by a three-point belt with a commercial wheelchair-anchored lap belt and surrogate shoulder belt.

BRODA RECOMMENDS THE TRAVERSA BE TRANSPORTED WITH SOME TILT ENGAGED FOR OPTIMAL SAFETY IN THE EVENT OF A CRASH.

The WC19 test was conducted using 48kph (30mph) and 20-g average impact conditions to determine the frontal impact response of the wheelchair.

The Traversa consists of transport specific features and consists of:

- Front Securement (left and right).
- Rear Securement (left and right).
- 6"x2" Front casters.
- Vehicle Restraint Seat Belt (conforms with WC19) and Anchor.

Use only Wheelchair Tie Down and Occupant Restraint Systems which meet the requirements of SAE J2249. Recommended Practice-Wheelchair Tie Downs and Occupant Restraint Systems for Motor Vehicles.

Chairs involved in vehicle accidents should be taken out of use immediately.

If a problem is detected with any of the Vehicle Transport Option components or any components of the chair, remove the chair from service and contact Broda immediately.

Vehicle Transport Components (from L-R): Front securement, rear securement.

Instructions

- 1) It is recommended users transfer to the vehicle seat and use the vehicle-installed restraint system.
- 2) The pelvic belt should be worn low across the front of the pelvis, so that the angle of the pelvic belt is within the preferred zone of 45° to 75° to the horizontal, or the optional zone of 30° to 45° to the horizontal as shown in figure 8.
- 3) Belt restraints should not be held away from the body by chair components or parts, such as the wheelchair armrests or wheels.
- 4) Upper torso belts should fit over the shoulders. as the wheelchair armrests or wheels.
- 5) Belt restraints should be adjusted as firmly as possible, consistent with user comfort.
- 6) Belt webbing should not be worn twisted in a manner that significantly reduces the area of contact of the belt with the occupant.

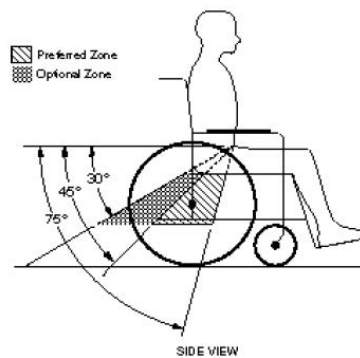
Figure 8 - Zones of preferred and optional angles for pelvic belts.

WARNINGS!

- 1) The chair should be used only for forward-facing seating in motor vehicles.
- 2) The chair should be used as indicated in the manufacturer's instructions.
- 3) The chair has been dynamically tested in a forward-facing mode with the ATD restrained by both pelvic and shoulder belts, and that both pelvic and upper torso belts should be used to reduce the possibility of head and chest impacts with vehicle components.
- 4) Only belt restraints that comply with the provisions of 4.9.2 through 4.9.5 and 5.2, and that have been dynamically tested in accordance with Annex A and 5.3, should be installed on the chair for use as a restraint in a motor vehicle.
- 5) Postural supports and belts may be used in a vehicle in addition to the occupant belt restraint, but should not be relied upon to replace occupant restraints that have been designed and tested for this purpose.

- 6) If chair is not equipped with a belt restraint that complies with 4.9.2 through 4.9.5 and 5.2, and that has been dynamically tested in accordance with Annex A and 5.3, or if chair user chooses not to use such a wheelchair-anchored –belt restraint, a vehicle anchored belt restraint should be used.
- 7) In order to reduce the potential of injury to vehicle occupants, chair-mounted trays not specifically designed for crash safety should be removed and secured separately in the vehicle.
- 8) Traversa should be consulted in case of questions about using chair for seating in a motor vehicle.
- 9) Alterations or substitutions should not be made to the chair structural and frame parts or components without consulting Traversa.
- 10) Recommended clear zones are in front and behind the chair user, as shown in figure frontal clear zones need to be larger when upper torso belts are not used.
- 11) The back support of the Traversa should not be reclined more than 30° to the vertical during vehicle travel unless necessary for the postural and medical needs of the occupant.
- 12) If it is necessary to recline the back support more than 30°, it is recommended that the upper shoulder-belt anchor point or D-ring is moved rearward so that the shoulder belt remains in contact with the shoulder and chest of the occupant.
- 12) Transportation in a large vehicle is recommended when the option exists.

4 INSPECTION AND FUNCTIONAL TESTING

4.1 Inspection

- 1) The cylinders that position the seat tilt, back recline, and independent leg rest elevation.
- 2) The vinyl straps used in the seat, back, leg rest, footrest and wings.
- 3) The cushions
- 4) The 3 black handles and cables.
- 5) The armrest height adjustment pins.

The visual inspection procedure should include the observation of any installed accessories. The visual inspection should be performed by the facility, or if in a private residence, by the individual responsible for the chair.

4.2 Functional Testing

We recommend that the chair should be tested for operation of the chair's functions without a passenger in the chair.

The testing may be as often as each use if there is any reason to be concerned about the possibility of increased wear or damage to the chair's components. At a minimum, in regular use the chair should be tested for functions as described in Section 3.

If the caregiver or maintenance department performing the functional testing believes that any function is not operating correctly, the chair should be taken out of service until a satisfactory functional test can be completed.

The caregiver or maintenance department performing the testing should be aware that the seat tilt, back recline, and footrest (leg rest) elevation operations will be more difficult without an occupant in the chair.

When performing repairs or maintenance, do not use lubricants that contain solvents. Solvents will damage many of the moving components in the chair. If necessary, a white, food grade grease (lubricant) may be used on the sliding components in the chair. Do not use spray lubricants on any part of the chair.

5 TECHNICAL INFORMATION

5.1 Specifications

Specifications Based on 20" Wide Traversa Transport Wheelchair

- 1. Back Recline:** Up to 90° (infinitely adjustable)
Trendelenburg position capable
- 2. Seat Tilt:** Up to 20° of seat tilt (infinitely adjustable)
- 3. Frame:** Powder coated steel
- 4. Seat/Back Surface:** Broda Comfort Tension Seating® Strapping is 1-1/2" wide, by 1/8" thick; contain fungicide, has shape memory retention, is fire retardant.
- 5. Arms:** Seat mounted arms
Removable and width adjustable
Adjustable armrest height
- 6. Leg Rest:** Leg rest articulates and extends with back recline.
Leg rest adjusts independently of back recline
Leg rest adjusts to any angle between horizontal and vertical
- 7. Footrest:** Flip down, ABS
- 8. Wings:** Adjustable and removable wings for upper lateral support.

9. Padding: Removable padding (shoulder bolsters, neck rest, upper & lower side pads) with removable vinyl cover, calf pad, sole pad). The 26 ounce healthcare vinyl fabric covering is manufactured to meet the following requirements: DIN 75 200/DIN 53 438; MVSS 302, M2;CAL 117, and Wyzenbeck. Heavy Duty Abrasion Test; anti-stain and anti-static finish; Cold crack of -10°F. Polyurethane foam meets CAL 117-2013.

Fully wrapped seat and back pads are covered with an 8 ounce soft vinyl which meets CAN2-162 flammability standard for hospital textiles. Polyurethane foam meets CAL 1172013.

10. Casters/Mags: 6" front casters,
14" rear, non-marking grey rubber tires.

11. Seat Depth: 18.5"

12. Seat Height: 22.5"

13. Seat Width: 20"

14. Overall Width: 27-5/8"

15. Overall Height: 50"

16. Weight Capacity: 450 lbs

17. Overall Weight: Approx. 175 lbs standard features

All dimensions are ± 0.5 ".

6 COMMON TROUBLESHOOTING AND REPAIR

Please have the chair serial number ready when contacting Traversa or your local area representative for parts or assistance for your chair. Serial number can be found on a label on the rear of the chair.

6.1 Troubleshooting for Gas Cylinders

The use of gas charged springs (cylinders) on Traversa products allows the caregiver to easily make adjustments to the tilt, recline, and leg rest with a minimal effort. The gas cylinders contain Nitrogen gas, it is not flammable nor is it toxic. The cylinders provide a lifting force which counterbalances the weight of the occupant thereby reducing the amount of weight the caregiver needs to lift.

Symptom	Problem	Solution
The chair is not reclining or tilting or leg rest is not moving when the handle is squeezed.	The pin in the end of the cylinder is not being depressed when the handle is pulled.	Ensure the cable is attached to the cylinder actuator. If cable is attached, see Adjustment Steps on next page.
The chair (tilt, recline or leg rest) seems to slowly slide out of position.	The pin in the end of the cylinder may be still partially depressed.	See Adjustment Steps on next page:

6.2 Adjustment Steps


Loosen the lock nut located on the stem of the gas cylinder next to the actuator (17mm or 11/16" wrench).

From the above instruction determine whether you want to wind the stem further into the actuator or further out of the actuator. Wind the stem further into the actuator if the chair is not tilting or reclining, (wind clockwise) and wind it further out of the actuator if the chair is slowly sliding out of position. (wind counter-clockwise).


- Wind the stem in the correct direction one half turn.
- Check the function of the cylinder and repeat the above step until the cylinder functions properly.
- Tighten the lock nut loosened in the first step.
- If adjusting the cylinder does not correct the problem, the cylinder may need to be replaced. Please contact Traversa at or your local area representative.

Note: When adjusting cylinders with tools that come in contact with the cylinder rods, a cloth or some other protective material must be used to protect the sliding surface of the rod. If the rod becomes scratched or marred it will damage the seal on the cylinder and the cylinder will fail.

Damaging a cylinder in this fashion voids the manufacturer's warranty.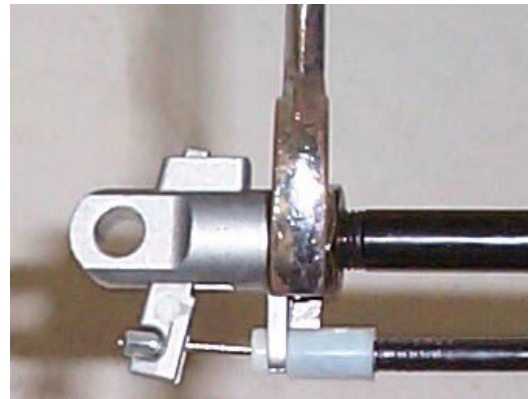

Note: Cylinder does not need to be removed from the chair to make this adjustment.

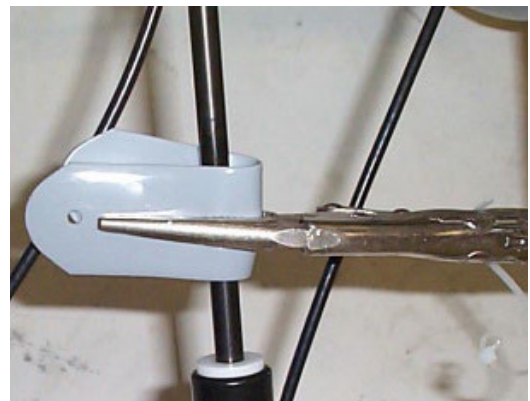MANUFACTURER WARRANTY

Effective 10/15/2018

From date of purchase Broda warrants the side frames and cross members located on the base frame, for the Expected Lifetime* of the original purchaser/user, when purchased from an Authorized Broda Dealer. *Expected Lifetime of the frame is 5 years.

In addition, Broda warrants its other product components, except the seat cushion and covers (which are not warranted), to be free from defects in materials and workmanship for a period of 2 years from the date of purchase, as indicated on the original purchase order. This warranty is non-transferable.

The warranty is subject to the following conditions:

A serial number or invoice number is required for warranty coverage. The warranty shall not apply to serial numbered items if the serial number has been removed or altered in any way.

Warranty is valid for "new" purchases only. All other items will be warranted at the sole discretion of Broda.

If within such warranty period any such product shall be proven to be defective through examination, inspection and testing as deemed necessary, such product shall be repaired or replaced at Broda's discretion. This warranty does not include onsite labor.

On request, defective parts must be returned to the factory, pre-paid and insured, within 30 days. Failure to do so, will result in the owner incurring the cost of the replacement parts. Photos of defective parts may be requested as an alternate to returning product.

Limitations and Exclusions:

This warranty does not cover damages that arise from improper handling, cleaning, maintenance, storage or negligent use. The warranty does not cover Broda's wheelchairs and chairs used in any way, other than the manner in which it was designed and recommended.

Modifications, incorrect assembly or installation to Broda products or the use of non-Broda products voids the warranty. Installation of parts that require the frame to be altered in any way, must be completed by a Broda Representative, unauthorized performances will void the warranty.

This warranty does not cover freight damage (see Damaged Freight Policy).

Broda retains the right to make product design and product application changes without notice.

The application of Broda products shall remain the responsibility of the purchaser or user. Please reference the model specific operating instruction manuals for safety requirements, inspectional and functional testing, technical information and more.

Warranty does not include on-site labor for the installation of warranty parts or warranty repairs. The owner may return to Broda products for warranty replacement or repair by shipping items prepaid and insured to the factory. Warranty completed at the factory includes both materials and labor. The decision to repair or replace parts is at the discretion of Broda. All returns to the factory require prior authorization from Broda.

MANUFACTURER WARRANTY

Effective 10/15/2018

WC-19 Transportation Certified Products:

Vehicle Transport Products and Options are available by factory install only.

It can NOT be retrofit to existing models or serviced in the field. All components of the vehicle transport packages are subject to specific maintenance requirements, to maintain the Broda warranty.

Due to the testing requirements of the vehicle transport chair, Broda warranty limitations apply to non-factory installed repairs. Warranty and maintenance repairs must be made by Broda authorized service technicians. Any replacements or repairs made by unauthorized person(s) will void this warranty and Broda assumes no liability on the performance of said components.

DO NOT modify the transport chair. Modifications may change the transport chair's structure and center of balance, which may increase the risk of serious injury to the user and void the warranty. If a problem is detected with any of the Vehicle Transport Product and Option components or any components of the chair, remove the wheelchair or chair from service and contact Broda immediately. Chairs involved in vehicle accidents should be taken out of use immediately.

**For any further questions regarding our Manufacturer Warranty please contact
Broda Customer Service at 1-844-552-7632 or customerservice@brodaseating.com**

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